

Nyxoah MRI Scanning Guidelines Europe



MRI Safety Information

The Genio® Implantable Stimulator (IS) is a Magnetic Resonance (MR) Conditional device which means that under MR environment the implant is safe within certain conditions.

This section contains important information regarding the Genio® IS and the conditions in which a patient implanted with the Genio® IS can safely undergo an MRI scan. MRI scans must be performed only as described in this section.

MRI Technician warnings and precautions

- › Genio® system external devices (such as the Genio® DP or AC) shall not be brought by the patient to the MRI scan room and used during the MRI scan. The use of the Genio® DP and AC during an MRI scan is prohibited.
- › Do not scan patients with fever.
- › During the MRI scan, continuously monitor the patient, notice any signs of anxiety and/or discomfort.
- › In case the patient feels uncomfortable pain or heating during the MRI scan, consider stopping the scan. Consider administration of anaesthetics to reduce patient's pain or discomfort.
- › The MRI scan should be conducted at least 6 weeks after patient implantation or revision surgery.

Potential risks associated with MRI scans

The Genio® IS device has been designed to minimize the potential adverse events that could result in patient harm. The potential MRI-related adverse events are listed below:

- › Implant heating causing damage to tissue in contact with the implant
- › Implant migration causing damage to tissue in contact with the implant
- › Implant migration causing the implant to be surgically removed (and replaced)
- › Unintended over stimulation causing damage to tissue in contact with the implant
- › Unintended stimulation causing discomfort due to electrical stimulation
- › Device malfunction causing the implant to be surgically removed (and replaced)
- › Diagnostic problems due to artifacts — shadowing on the MRI image in the vicinity of the implant causing loss or disturbance of diagnostic information

MRI Scan Conditions

Non-clinical testing has demonstrated the Genio® IS is MR Conditional. A patient with this device can be safely scanned in an MR system meeting the following conditions:

Static magnetic field of 1.5 Tesla and 3 Tesla, with:

- › Maximum spatial field gradient of 1000 G/cm (10 T/m)
- › Maximum force product of 18,000,000 G²/cm (18 T²/m)
- › Maximum switched gradient slew rate per axis of 200 mT/m/ms
- › Maximum switched gradient amplitude per axis of 45 mT/m
- › Full body MRI scan in Normal Operation Mode for 15 minutes duration

Scan Condition 1 (1.5T)

For the torso (excluding thorax) and lower regions landmark position a maximum Whole Body Averaged (WBA) specific absorption rate (SAR) of 2 W/kg (Normal Operating Mode)

Under the scan conditions defined above, for a continuous scanning of 30 minutes, the Genio® Implantable Stimulator is expected to produce a maximum temperature rise of less than:

- › 1.05°C (2 W/kg, 1.5 Tesla) RF-related temperature increase.
- › 0.5°C (42 T/SRMS) gradient magnetic field induced temperature increase.

Scan Condition 2 (1.5T)

For the head/neck/thorax landmark position a Head Averaged (HA) specific absorption rate of 3.2 W/kg (Normal Operating Mode)

Under the scan conditions defined above, for a continuous scanning of 15 minutes, the Genio® Implantable Stimulator is expected to produce a maximum temperature rise of less than:

- › 5.14°C (2 W/kg WBA and 3.2 W/kg HA, 1.5 Tesla) RF-related temperature increase.
- › 0.5°C (42 T/SRMS) gradient magnetic field induced temperature increase.

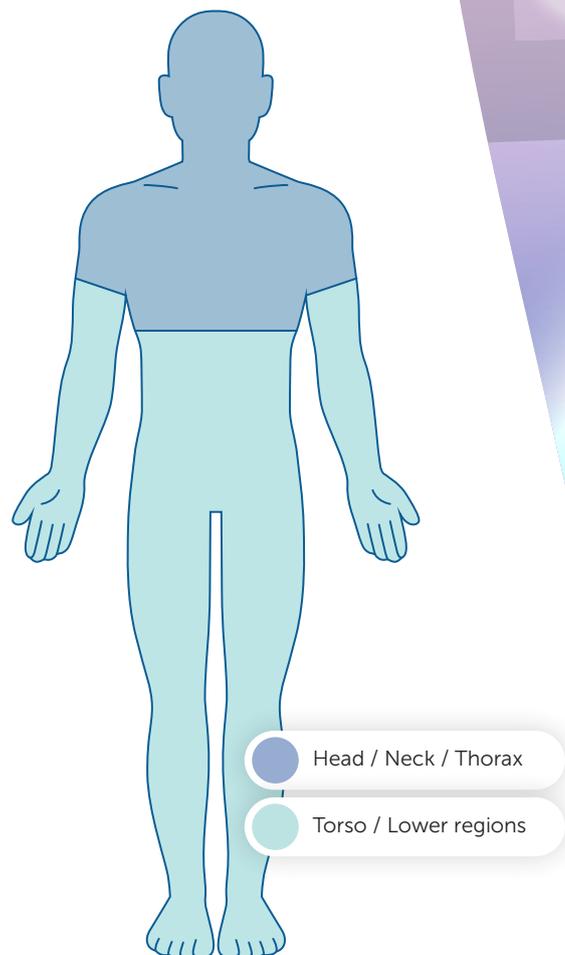


Figure 1: Scan Condition (1.5T and 3T)

Scan Condition 3 (3T)

For the torso and lower regions (excluding thorax) landmark position a maximum Whole Body Averaged (WBA) specific absorption rate (SAR) of 2 W/kg (Normal Operating Mode)

Under the scan conditions defined above, for a continuous scanning of 30 minutes, the Genio® Implantable Stimulator is expected to produce a maximum temperature rise of less than:

- › 1.2°C (2 W/kg, 3 Tesla) RF-related temperature increase.
- › 0.5°C (42 T/SRMS) gradient magnetic field induced temperature increase.

Scan Condition 4 (3T)

For the head/neck/thorax landmark position a Head Averaged (HA) specific absorption rate (SAR) of 3.2 W/kg (Normal Operating Mode) and a Whole Body Averaged (WBA) of 2 W/kg (Normal Operating Mode)

Under the scan conditions defined above, for a continuous scanning of 15 minutes, the Genio® Implantable Stimulator is expected to produce a maximum temperature rise of less than:

- › 5.5°C (2 W/kg WBA and 3.2 W/kg HA, 3 Tesla) RF-related temperature increase.
- › 0.5°C (42 T/SRMS) gradient magnetic field induced temperature increase.

MR-induced malfunction testing with device exposure to the static field B₀, the switched gradient field (dB/dt), the RF field (B₁, E) and combined fields in 1.5 Tesla and 3 Tesla MR systems passed all tests in compliance with relevant standards.

In non-clinical testing, the image artifact caused by the device extends approximately 49.4 mm from the Genio® Implantable Stimulator when imaged with a gradient echo pulse sequence and a 3 Tesla MR system.

All rights reserved © 2021 Nyxoah SA. All content of these Instructions for Use, including the texts, trademarks, service marks, logos, illustrations, photos, graphics, design etc., are the property of Nyxoah SA. Nyxoah SA owns all rights with respect to any of their trademarks, service marks, logos, and copyrights appearing in these Instructions for Use.

Patented and design protected technology.

CE marked since 2019.

Doc ID: EXT-0001336 Rev01

Manufacturer Information:

Nyxoah SA
Rue Edouard Belin 12
1435 Mont-Saint-Guibert
Belgium
info@nyxoah.com
+32 10 22 23 55

Nyxoah[®]